Organ Recovery systems

Chicago
2570 E. Devon Avenue
Des Plaines, IL 60018
tel 847.824.2600
toll free in US 866.682.4800
www.organ-recovery.com

Brussels
Da Vincilaan Box 6
1935 Zaventem
Belgium
tel 32 (0)2 715.0000
www.organ-recovery.com

European Authorized Representative
Emergo Europe
Molenstraat 15
2513 BH The Hague
The Netherlands

Representante Autorizado para Europa
Emergo Europe
Molenstraat 15
2513 BH The Hague
The Netherlands

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An ISO 13485 certified company
© Organ Recovery Systems, Inc. Printed in the USA

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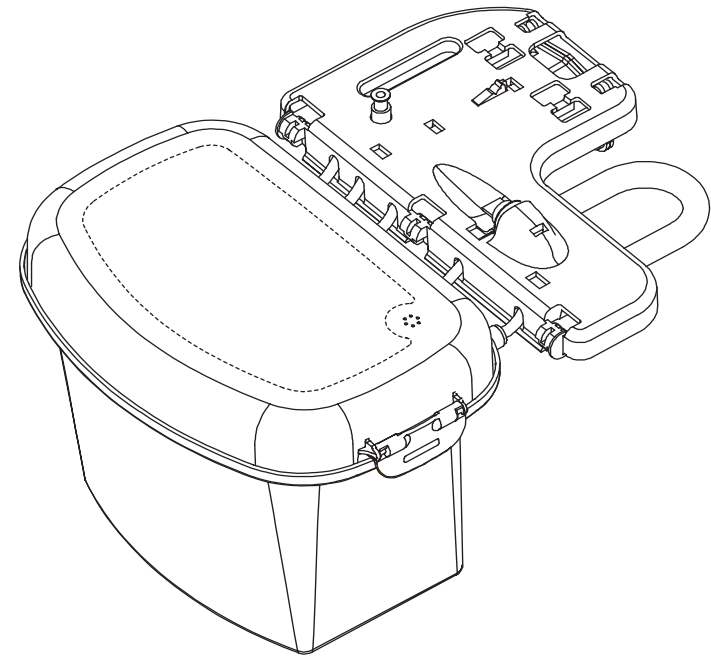
Organ Recovery systems

LifePort™ Kidney Transporter Perfusion Circuit

LifePort™ disposable perfusion circuit for use only with
LifePort™ perfusion equipment

Manufactured for:
Organ Recovery Systems
2570 E. Devon Avenue
Des Plaines, IL 60018
USA

847.824.2600

Indication: LifePort™ kidney perfusion circuit is for use only with the LifePort™ Kidney Transporter. The LifePort™ is intended to be used for the continuous hypothermic machine perfusion of kidneys for the preservation, transportation and eventual transplantation into a recipient.

Warnings:

-  Federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner)
-  Use universal precautions (e.g., gloves, masks, gowns, goggles or equivalent eye protection, biohazard bags) in handling the kidney, and in handling and disposing of the perfusion circuit and perfusate to prevent the possible transmission of pathogens to medical personnel and patients.
-  While preparing and working within the perfusion circuit follow hospital or physician-prescribed aseptic protocols. The single practitioner, working alone must pay special attention to maintain these conditions and may need to modify these procedures.
-  Visually check the perfusion circuit before use. Do not use if parts are loose, cracked, broken, or liquid is leaking. Check that all fittings are tight before installation.
-  The sterilized perfusion circuit is intended for a single use. After use, the entire circuit should be disposed in accordance with local guidelines for biohazardous waste.

Instructions for Use:

Before getting started, prepare the LifePort™ Kidney Transporter as described in the operator's manual. Also prepare KPS-1® or another solution approved for machine perfusion of kidneys. Maintain perfusate at appropriate temperature, approximately 4-6 °C.

Note: Grey boxes highlight actions performed within the sterile field.

1. Prepare a sterile field on a work table and introduce all necessary materials.

This may include this perfusion circuit, cannula, sterile drape, instruments, and sterile gloves. Organ Recovery Systems provides sterilized perfusion circuits, cannulas, and drapes, in packaging that enables them to be either dropped onto the table or handed off to a scrubbed-in assistant. Visually inspect all material for damage; discard and replace if necessary.

2. Fill cassette with cooled perfusate.

A gowned and gloved person should remove the perfusion circuit from the white packaging tray and set the cassette lids and kidney cradle carefully aside. Always set the inner cassette lid face-down on a sterile surface. Position the cassette so that a person outside the sterile field can decant 1-liter cold perfusate into the cassette housing, while maintaining the cassette and lids within the sterile field. This procedure can be modified to accommodate a single practitioner.

3. Seal the perfusion circuit before transferring to the transporter.

Both cassette lids should be replaced and secured by the gowned and gloved person. Check the tightness of all fittings. Then hand the filled and sealed perfusion circuit to a person outside the sterile field.

Once the perfusion circuit is removed from the sterile field, only the inner surfaces should be considered sterile and all further procedures should be carried out with this in mind.

4. Load the perfusion circuit into the transporter.

Transfer the circuit to the transporter. Lower the cassette into its mating cavity in the ice bucket. Carefully assemble the tube frame onto the pump deck. The tube frame is assembled with the tube frame UPRIGHT, perpendicular to the pump deck. Only when the tube frame is perpendicular will the hinges slide into the receivers on the pump deck. The tube frame will not assemble properly if it loaded horizontally.

5. Secure the tube frame onto the pump deck.

After the hinges are located, the tube frame is rotated down over the pump deck. Stretch the pump tubing around the head of the infusion pump, close and latch the pump raceway, and rotate the pump deck locking arm 90 degrees and snap into place. Connect the pressure sensor cable from the pump deck to the pressure sensor connector on the tube frame.

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6. Prime the perfusion circuit.

Follow the procedures outlined in the LifePort™ Kidney Transporter Operator's Manual to energize the transporter and prime the circuit. To maintain a cold environment, the transporter cover should be replaced after the circuit is primed.

7. Cannulate the kidney.

Cannulation is the key step in using the transporter. Various types and sizes of cannula are available. The following steps need to be considered and performed in choosing the proper cannula and to achieve successful cannulation.

- A – minimize trauma to the renal artery
- B – Perfuse all of the renal arteries
- C – Seal all leaks

Refer to the operator's manual and cannula instructions for use for additional detail.

8. Secure the kidney.

Place the cannulated kidney into the kidney cradle. Snap the cannula into the cannula mount. Adjust the mount by squeezing the sides and sliding to a height where the artery will not kink. Do not overstretch the artery. Make sure that the cannula is securely attached to the artery, allowing it to remain in place during perfusion and transportation. Visually inspect the renal artery ensuring there are no twists or occlusions. Correct any twists or occlusions before starting the perfusion procedure.

9. Prepare the transporter with the sterile drape.

A person outside the sterile field should remove the transporter cover and the outer cassette lid, being careful not to touch the inner cassette lid or underside of the outer cassette lid. Place the outer cassette lid upside-down onto a clean table where it will not be disturbed.

The gowned and gloved practitioner should drape the transporter. The sterile drape gasket should be secured and the drape unfolded; covering the transporter and exposing the inner cassette lid. Refer to the operator's manual or sterile drape instructions for use for additional detail. Remove the inner cassette lid and place it in the sterile field.

10. Transfer the kidney to the transporter.

Carefully transfer the kidney and cradle from the sterile field to the draped transporter. Once at the transporter, lower the cradle into the cassette. Be careful to avoid catching the infuse line between the cradle and cassette.

The organ is now ready to be perfused. Follow the instructions within the operator's manual to properly prime the infuse line, cannula, and kidney. Follow the instructions to begin infusion.

Continue to follow aseptic protocol while replacing the inner cassette lid and removing the sterile drape.

Replace the outer cassette lid and transporter cover. The transporter is now ready to transport.

Storage condition: For best results, store at room temperature. Avoid excessive heat and humidity. Sterile unless package is damaged or open.

Contact: For technical assistance or to re-order please contact Organ Recovery Systems at 866.682.4800 (toll free in US) or 1.847.824.2600.

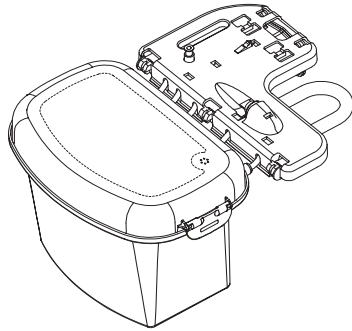
Caution: Federal and international law restricts this device to sale by or on the order of a physician or licensed practitioner.

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