

Flying LifePort Kidney Transporter (LKT) in Cabin of Commercial Aircraft

Checklist

Documents to accompany LKT

1. OPTN Identifiers
2. Copy of itinerary
3. Contact numbers for OPO and ORS 24/7 Perfusion Helpline number
4. Copy of TSA contact email from both airports
5. LKT battery specifications

Prior to Arriving at Outgoing Airport

- ☐ Understand basic LKT troubleshooting
- ☐ Swap LKT if necessary
- ☐ Ensure LKT has adequate ice prior to leaving OPO
- ☐ Make sure LKT batteries are charged
- ☐ Silence LKT alerts by turning Audible Alert Switch to "O" with a flathead screwdriver

At Outgoing Airport

- ☐ Bring LKT to airline ticket counter to verify documentation
- ☐ Send LKT through X-ray, if required. If any issues arise, request to speak to TSA Point of Contact

On Outgoing Plane

- ☐ Secure LKT in second seat and notate time of boarding

After Landing on Outgoing Plane

- ☐ Transport LKT to transplant hospital

At Airport for Return Flight

- ☐ Bring LKT to airline ticket counter to verify documentation
- ☐ Send LKT through X-ray, if required. If any issues arise, request to speak to TSA Point of Contact



On Return Flight

- ☐ Secure LKT in second seat

After Return Flight

- ☐ Return LKT to home base
- ☐ Resume audible alerts on LKT by turning Audible Alert Switch to "I" with a flathead screwdriver